

How to use the Bathmate Hydromax


Bathmate STEP 01



Loosen up in a warm shower



Bathmate STEP 02

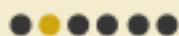

Fill pump with water



Press in pressure valve

Keep finger over valve whilst filling



Bathmate STEP 03



Place penis inside pump



Form a tight seal with pump and body



Bathmate STEP 04



Pump



Push towards pubic bone

Penis is drawn up the tube

Water is ejected from the valve



Pressure starts erection

BATHMATE HYDROMAX



Repump

Leave on for 15 mins

Bathmate STEP 05



Safely remove pump



Compress main valve to release pressure.



Bathmate STEP 06



Optional shower strap

Hand free shower strap

Easy use during shower

